

# Natalin™

Tri-Layer Amniotic Membrane Allograft

Natalin is a tri-layered, decellularized, dehydrated human amniotic membrane (DDHAM) that provides a protective cover from the surrounding environment. As a biological membrane covering, Natalin provides a barrier to protect the underlying tissue and preserve tissue plane boundaries.

## Easy to apply

1. Remove from the package and use sterile atraumatic forceps to remove Natalin from its inner peel-pouch.
2. Apply Natalin over the defect—no specific orientation is required.
3. Adheres well without sutures.

Please see Package Insert for full instructions.


## Minimal application time

- ◆ No preparation—no thawing, rinsing, or soaking required
- ◆ Flexible matrix—easily conforms to the surface, resulting in no additional bulk to site
- ◆ No orientation—Natalin can be applied with either side facing the tissue
- ◆ Adheres without sutures—or Natalin can be sutured, taped, stapled, or glued, as determined by the surgeon

## Maximum convenience

- ◆ 10-year shelf-life eliminates need for pre-ordering
- ◆ Room temperature storage—no refrigeration necessary



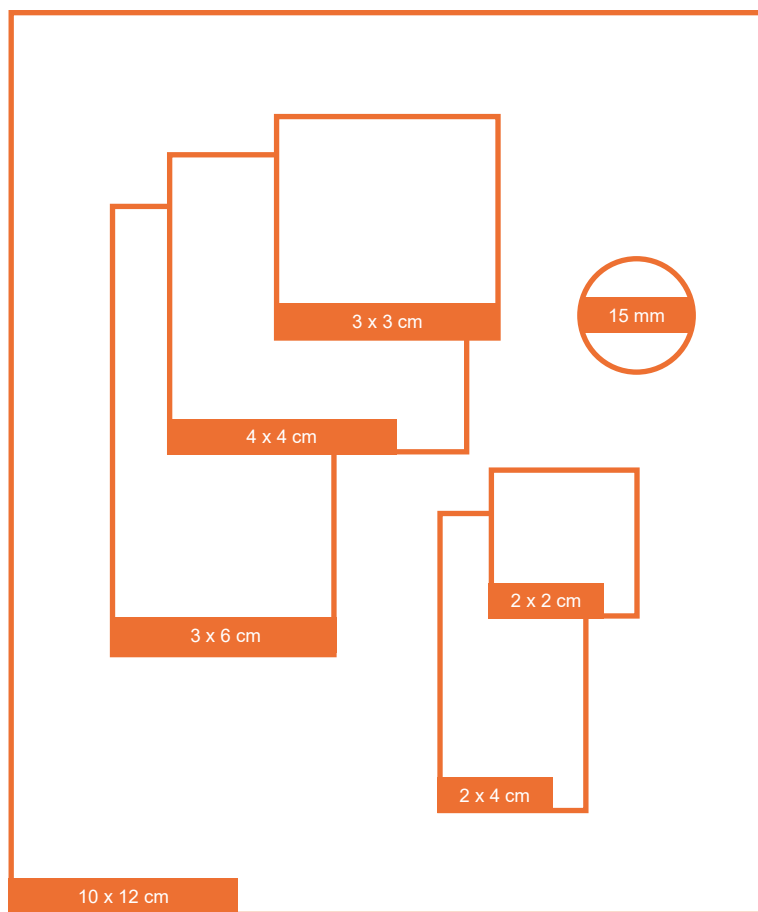

## Available in multiple sizes for application flexibility

15 mm	Product Code: DHAM015N
2 x 2 cm	Product Code: DHAM022N
2 x 4 cm	Product Code: DHAM024N
3 x 3 cm	Product Code: DHAM033N
4 x 4 cm	Product Code: DHAM044N
3 x 6 cm	Product Code: DHAM036N
10 x 12 cm	Product Code: DHAM012N

Natalin™ is an allograft intended for use as a biological membrane covering that provides the extracellular matrix. As a barrier membrane, Natalin is intended to protect the underlying tissue and preserve tissue plane boundaries. Indications include, but are not limited to, surgical covering, wrap or barrier, application to partial- and full-thickness, acute and chronic wounds (such as, traumatic and complex wounds, burns, surgical and Mohs surgery sites; and diabetic, venous, arterial, pressure and other ulcers), including wounds with exposed tendon, muscle, bone or other vital structures. Natalin is contraindicated in patients with a known hyper- sensitivity to Natalin. If a patient has an adverse reaction related to the use of Natalin, immediately discontinue its use. Natalin should not be used on clinically infected wounds. The pouch contents are sterile if the pouch is unopened and undamaged. Do not use if package seal is broken. Discard material if mishandling has caused possible damage or contamination. Do not resterilize. Natalin must be used prior to the expiration date on the product pouch. Natalin should not be used together with a collagenase product on the wound.



For product information or adverse reaction reporting, telephone 1-844-963-2273. Please refer to the Natalin Package Insert for complete product information. Natalin™ is a trademark of Celularity Inc. ©2025 Celularity Inc. All rights reserved NAT-2025-0001